

Ketac™ Cem

Glass Ionomer Luting Cement

72

Glass ionomer luting cement in the Aplicap™- or Maxicap™ System from 3M ESPE.

Indications

- Inlays / Onlays
- Crowns and bridges
- Pins / posts
- Orthodontic appliances
- Linings

Advantages

- Fluoride release
- Radiopaque
- The Aplicap™ / Maxicap™ System from 3M ESPE offers simple handling, constant working properties and reliable quality.

Instructions for Use

- During the process of activating, push activator lever down
- Keep lever in this position for at least 2 sec.
- **Mixing time**
 - Rotomix: 0:08 min.
 - Capmix: 0:10 min.
- **Working time (including mixing):**
 - Ketac Cem Aplicap 3:00 min.
 - Ketac Cem Maxicap 3:00 min.
- **Setting time (from begin of mix):**
 - Ketac Cem Aplicap 7:00 min.
 - Ketac Cem Maxicap 7:00 min.



Introductory Kits Ketac Cem Aplicap and Maxicap

Type of package:	Item number
Introductory Kit Ketac Cem Aplicap	56053
50 Aplicap Capsules	
1 Aplicap Activator	
1 Aplicap Applier	
Introductory Kit Ketac Cem Maxicap	56015
50 Maxicap Capsules	
1 Maxicap Activator	
1 Maxicap Applier	
Refills	
50 Aplicap Capsules	56060
50 Maxicap Capsules 56020	
1 Aplicap Activator	73040
1 Aplicap Applier	73050
1 CombiPack Aplicap Activator / Applier	37160
1 Maxicap Activator	73060
1 Maxicap Applier	73070
1 CombiPack Maxicap Activator / Applier	37180