


RelyX™ Veneer Cement

Light curing resin cement for veneers

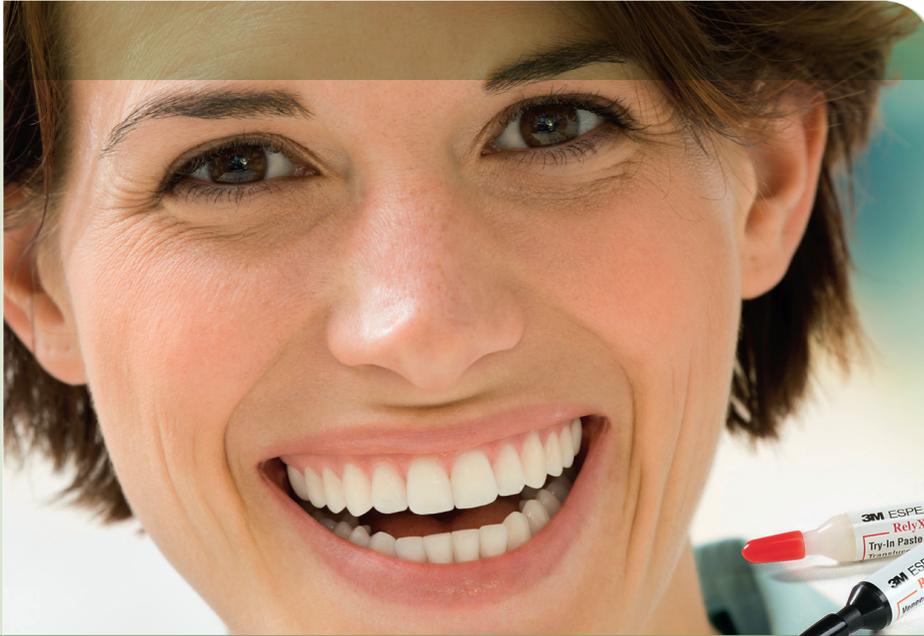

Easy to use and provides excellent aesthetic results.

Indications:

Permanent cementation of:

- Porcelain
- Ceramic and
- Composite veneers

Clinical Technique:



Photos courtesy of Dr. C. Sabrosa, Brazil

The simplicity you want and reliability you need.

RelyX™ Veneer Cement is a veneer cement system with a customer-preferred colour shading system that is easy to use and clean up.

RelyX Veneer Cement provides simple and efficient bonding of porcelain, composite and ceramic veneers. Combined with the 3M™ ESPE™ RelyX™ Try-In Pastes, the cement provides a simple and efficient way to perform veneer procedures.

- Optimal colour match of the water-soluble try-in pastes to the cured cement shade
- Excellent colour stability
- Ideal viscosity of the cement for easy insertion and positioning of the veneer



3M ESPE

Features and Benefits:

Customer-preferred delivery and shading system is easy to use.

Try-in pastes are water soluble and residue-free and offer excellent colour match to the final cured cement for better patient satisfaction.

Easy to clean up.

Ideal, non-slump cement viscosity prevents veneer drift and is easy to dispense, apply and seat.

RelyX™ Veneer Cement's excellent adhesion to dentine and enamel reduces the potential for microleakage and marginal staining.

Light-cure only, resin-based cement provides exceptional colour stability. Additionally, the cement cures with a wide range of light sources.

Available in six commonly used and preferred shades: Translucent, White Opaque, A1/Light Yellow*, A3 Opaque/ Yellow Opaque and B0.5/White.



Fast and easy bonding with Scotchbond™ Universal.

Scotchbond Universal is a single-bottle adhesive solution by 3M ESPE. It reliably bonds to all surfaces, including enamel, dentine, porcelain, composites and ceramic. Scotchbond Universal also functions as a silane primer for glass ceramic and composite veneers. This eliminates the need for a separate primer and more importantly reduces your inventory.

Ordering Information

Item No.	Product Information
	RelyX™ Veneer Cement
8716	RelyX™ Veneer Cement Intro Kit: 1 syringe cement Translucent 3g; 1 syringe cement White Opaque 3g; 1 syringe cement A1/Light Yellow 3g; 1 syringe cement A3 Opaque/Yellow Opaque 3g; 1 syringe cement B0.5/White 3g; 1 syringe try-in paste Translucent 2g; 1 syringe try-in paste White Opaque 2g; 1 syringe try-in paste A1/Light Yellow 2g; 1 syringe try-in paste A3 Opaque/Yellow Opaque 2g; 1 syringe try-in paste B0.5/White 2g; 1 Scotchbond™ Universal 5 ml vial; 1 Scotchbond™ Universal Etchant 3ml syringe; 25 Etchant Tips; 48 White Mix Wells; 50 Scotchbond™ Microbrushes; Step-by-Step Technique Guide
	Scotchbond™ Universal Adhesive
41258	Scotchbond™ Universal Adhesive Refill: 1 Scotchbond™ Universal 5 ml vial

Item No.	Product Information
	Cement Refills — Syringe
7614TR	Translucent Shade
7614W0	White Opaque Shade
7614A1	A1/Light Yellow Shade
7614A3	A3 Opaque/Yellow Opaque Shade
7614B0.5	B0.5/White Shade
	Try-In Paste Refills — Syringe
7614TRT	Translucent Shade
7614W0T	White Opaque Shade
7614A1T	A1/Light Yellow Shade*
7614A3T	A3 Opaque/Yellow Opaque Shade
7614B0.5T	B0.5/White Shade

*Australia only.



RelyX™ Veneer Cement Intro Kit (8716)

3M ESPE

3M Australia Pty Limited

ABN 90 000 100 096
 Building A, 1 Rivett Rd
 North Ryde NSW 2113
 1300 363 878
 www.3MESPE.com.au

3M New Zealand Limited

94 Apollo Drive
 Rosedale
 Auckland 0632
 Ph: 0800 80 81 82
 www.3MESPE.co.nz

3M, ESPE, L-Pop, RelyX and Scotchbond are trademarks of 3M © 2014, 3M. All rights reserved. PB7081/1014